



SIQURA™ ULTRAX

HOSPITAL GRADE SURFACE DISINFECTANT

ONE STEP FAST ACTING CLEANER DISINFECTANT

SIQURA™ UltraX HOSPITAL GRADE SURFACE DISINFECTANT is a premium, low alkaline - low odour hard surface bactericidal, yeasticidal and virucidal* disinfectant for use under dirty conditions in medical areas and other environments.

SIQURA™ UltraX is fast acting and is effective against SARS-CoV 2 in 60 seconds and a range of pathogens in 2 minutes (‡unless indicated).

SIQURA™ UltraX is suitable for Healthcare, Life science, Institutional and Janitorial environments where fast contact times, low corrosive single step cleaning and disinfection solution is desired.

‡ For efficacy against SARS-CoV-2 allow surface to dry for 60 seconds. For efficacy against Serratia marcescens, Norwalk virus (Norovirus) and Feline Calicivirus allow surfaces to dry for 5 minutes. For efficacy against Murine Norovirus (MNV) allow surfaces to dry for 10 minutes.

**1
STEP**

Rapid acting, single step cleaning and broad spectrum Hospital Grade disinfection*



Effective against a broad range of Multi Drug Resistant pathogens*



Effective in clean and dirty conditions



Suitable for use on most surfaces (Non corrosive)



Low pH formulation

SIQURA™ ULTRAX HOSPITAL GRADE SURFACE DISINFECTANT

SIQURA™ UltraX spray is suitable for use as a single step cleaner and disinfect.

IT'S AS EASY AS 1-2-3



- 1** Spray entire surface until wet. Allow surface to air dry for 2 minutes[†].
- 2** Allow to air-dry. Do not wipe surface as this will remove the coating before it's had a chance to bond. Once dry, the antibacterial active is locked onto the surface.
- 3** Surface is now secured and will defend against bacterial contamination for up to 24hrs on high touch areas and 30 days on low touch areas, unless heavy abrasion occurs.

- ✓ One Step Clean and Disinfect
- ✓ Rapid Action
- ✓ Alternative to Alcohol or Bleach
- ✓ Non-oxidative
- ✓ Non-flammable
- ✓ Low Odour
- ✓ Low streak formula
- ✓ Low pH

IDEAL FOR USE WITHIN

Healthcare facilities, Aged Care facilities, Lifescience laboratories, Janitorial, Institutional environments, Schools and Early Learning environments, Public transportation, Quick Service Restaurants, Public areas.



SIQURA™ ULTRAX PRODUCT RANGE

CODE	PRODUCT	PACK SIZE
SQ-453	UltraX 500ml Re-Usable Bottle	Box 6
SQ-387	UltraX Biodegradable Wipes 80's	Box 12
SQ-476	UltraX Lint Free Wipes 100's	Box 12
SQ-409	UltraX 5ltr Jug	Box 2
SQ-447	UltraX 20ltr Drum	1 Drum

[†] For efficacy against SARS-CoV-2 allow surface to dry for 60 seconds. For efficacy against Serratia marcescens, Norwalk virus (Norovirus) and Feline Calicivirus allow surfaces to dry for 5 minutes. For efficacy against Murine Norovirus (MNV) allow surfaces to dry for 10 minutes.

