



SIQURA™ NeutralX

SURFACE CLEANER SPRAY

Powerful neutral pH plant based biodegradable detergent cleaner

SIQURA™ NeutralX is a specifically formulated neutral detergent cleaner designed to clean surfaces and break down proteins, fats, grease and oils that contaminate a surface. NeutralX also breaks down biofilms that form a barrier to effective surface cleaning. NeutralX is SLS, SLES and paraben free and is readily biodegradable, so it's kinder to you and the environment. NeutralX is residue free and is safe to use on most surfaces including aluminium, steel, plastic and fabrics.

NeutralX Detergent Spray is a Class 1 Medical Device intended for cleaning healthcare surfaces and other Medical Device surfaces.



Powerful pH neutral detergent



Breaks down biofilms, proteins, fats, grease and oils



Ready to use, industrial refill trigger bottle



Suitable for cleaning healthcare and other medical device surfaces (non-corrosive)



Biodegradable plant based formulation

DIRECTIONS FOR USE

Shake well and hold the spray bottle upright, approx. 200 mm from the surface. Spray the entire surface area and wipe clean using an S pattern from one side of the surface to the other. Rinse with potable water following application to food contact surfaces.

To refill NeutralX refillable spray bottle

Step 1: Fill the bottle with clean water to Line 1.

Step 2: For general cleaning, add 20ml of Siquira NeutralX Detergent Concentrate to Line 2 or for heavy grease and soil removal, add 30ml of Siquira NeutralX Detergent Concentrate to Line 3.

Step 3: Fill the bottle up to the 500ml line marked on the neck of the bottle with clean water.

Step 4: Spray the entire surface area and wipe clean using an S pattern from one side of the surface to the other. Rinse with potable water following application to food contact surfaces.

1 Add water to line 1



2 Add desired amount of concentrate & add more water to 500ml line



3 Shake to mix and spray 200mm from surface



4 Wipe clean in an S pattern



FEATURES

- pH Neutral
- Powerful formula breaks down protein & biofilms
- ARTG #427136
- Biodegradable plant based surfactants
- Alcohol and disinfectant free
- Made in Australia

RANGE OF USE

- ✓ HEALTHCARE
- ✓ MEDICAL VEHICLES AND TRANSPORT
- ✓ AGED CARE
- ✓ MANUFACTURING
- ✓ DENTAL
- ✓ RETAIL
- ✓ EDUCATION
- ✓ COMMERCIAL & OFFICE
- ✓ HOSPITALITY
- ✓ PUBLIC TRANSPORT

NeutralX DETERGENT SPRAY is derived from ethoxylated fatty acid esters of coconut oil, boasts low aquatic toxicity and minimal irritancy while delivering strong emulsification and detergent capabilities.

SIQURA™ NeutralX SPRAY

CODE	PRODUCT	PACK SIZE
SQ-428	Siquira NeutralX 500ml Re-fillable bottle (Empty)	(10 per box)
SQ-474	Siquira NeutralX 500ml Re-fillable bottle (Filled)	(10 per box)
SQ-425	Siquira NeutralX 5ltr jug	(2 per box)
SQ-427	Siquira NeutralX 20ltr drum	

