



# SIQURA™ NeutralX

## SURFACE CLEANER CONCENTRATE

**Powerful neutral pH plant based biodegradable detergent concentrate**

**SIQURA™ NeutralX DETERGENT WIPES ARE SPECIFICALLY FORMULATED NEUTRAL DETERGENT WIPES** designed to clean surfaces and break down proteins, fats, grease and oils that contaminate a surface. NeutralX Wipes also break down biofilms that form a barrier to effective surface cleaning. NeutralX Wipes are SLS, SLES and paraben free and are readily biodegradable, they're kinder to you and the environment. NeutralX Wipes are residue free and is safe to use on most surfaces including aluminium, steel, plastic and fabrics.

NeutralX Detergent Wipes are a Class 1 Medical Device intended for cleaning healthcare and Medical Device surfaces.



**Powerful pH neutral detergent**



**Breaks down biofilms, proteins, fats, grease and oils**



**Dilutable concentrate**



**Suitable for cleaning healthcare and other medical device surfaces (non-corrosive)**



**Biodegradable plant based formulation**

## DIRECTIONS FOR USE

NeutralX Detergent Concentrate can be diluted for mopping, wiping or spray applications. For general use, in a suitable container dilute 40ml of concentrate into 1 litre of water. For heavy grease and soil removal, dilute 60ml of concentrate into 1 litre of water.

### To refill NeutralX refillable spray bottle

**Step 1:** Fill the bottle with clean water to Line 1.

**Step 2:** For general cleaning, add 20ml of Siquira NeutralX Detergent Concentrate to Line 2 or for heavy grease and soil removal, add 30ml of Siquira NeutralX Detergent Concentrate to Line 3. Fill the bottle up to the 500ml line marked on the neck of the bottle with clean water.

**Step 3:** To apply, shake spray bottle well and hold upright, approx. 200mm from the surface.

**Step 4:** Spray the entire surface area and wipe clean using an S pattern from one side of the surface to the other. Rinse with potable water following application to food contact surfaces.


**1** Add water to line 1



**2** Add desired amount of concentrate & add more water to 500ml line



**3** Shake to mix and spray 200mm from surface

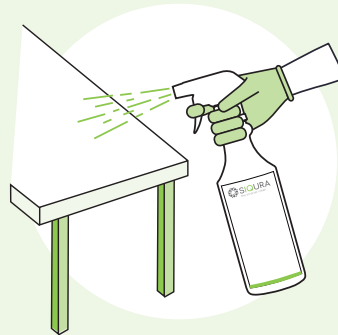

**4** Wipe clean in an S pattern



## FEATURES

- pH Neutral
- Dilutable concentrate
- Powerful formula breaks down protein & biofilms
- ARTG #427136
- Biodegradable plant based surfactants
- Alcohol and disinfectant free
- Made in Australia

## RANGE OF USE

- ✓ HEALTHCARE
- ✓ MEDICAL VEHICLES AND TRANSPORT
- ✓ AGED CARE
- ✓ MANUFACTURING
- ✓ DENTAL
- ✓ RETAIL
- ✓ EDUCATION
- ✓ COMMERCIAL & OFFICE
- ✓ HOSPITALITY
- ✓ PUBLIC TRANSPORT

NeutralX DETERGENT CONCENTRATE is derived from ethoxylated fatty acid esters of coconut oil, boasts low aquatic toxicity and minimal irritancy while delivering strong emulsification and detergent capabilities.

## SIQURA™ NeutralX DETERGENT CONCENTRATE

CODE	PRODUCT	PACK SIZE
SQ-425	Siquira NeutralX 5ltr jug	(2 per box)
SQ-427	Siquira NeutralX 20ltr drum	
SQ-428	Siquira NeutralX 500ml Re-fillable bottle (Empty)	(10 per box)
SQ-474	Siquira NeutralX 500ml Re-fillable bottle (Filled)	(10 per box)

